

Atlantis® CustomBase solution

Extraoral cementation procedure for angulated screw access

Atlantis Core File now available for released angulated screw access interfaces*



1

Clean all parts. Insert the screw into the abutment **BEFORE** cementation of the crown. The screw cannot go through the screw access hole after cementation.
Note: Try-in before cementation is recommended.



2

The screw head and counterbore must be protected from cement using suitable materials that are easy to remove before the cement has fully cured.



3

Apply cement onto the abutment and fully assemble the crown and abutment.



4

Carefully remove and clean excess cement from the material junction, the screw access hole and inside the counterbore. The screw is now captured inside the solution and use the screwdriver to confirm the screw is free to move inside.



5

Install the restoration using the Atlantis Angulated Screw Access Screwdriver.



Installation procedures.
See for yourself.



Handling procedures.
See for yourself.

*Refer to the Atlantis abutments implant compatibility chart for latest update of available assortment for angulated screw access interfaces.

How to order

Screwdriver and Driver Handle

Atlantis Angulated Screw Access Screwdriver is required during installation. Order the screwdriver and your choice of driver handle in Dentsply Sirona Implants eCommerce or contact customer service.


Screwdriver for Atlantis CustomBase solution

Multiple-use, stainless steel, non-sterile



| | | | |
|-----------------|-------|-------|-------|
| Total length mm | 18 | 24 | 32 |
| Order no. | 36030 | 36031 | 36032 |

The screw will be captive inside the solution - Do NOT remove

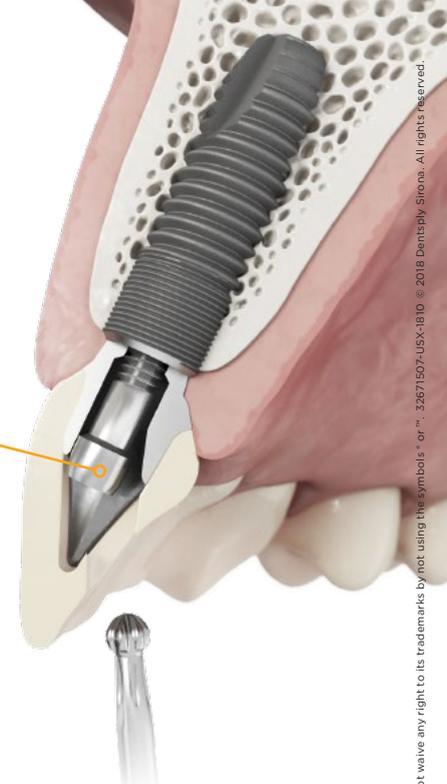

Use Atlantis Screw Access Screwdriver together with the selected Driver Handle. Driver Handles are available in the Dentsply Sirona Implants eCommerce and can currently not be order in Atlantis WebOrder. Some of the handles are not compatible to use for wrench connections and only for torque with finger force.

Driver Handle for Dentsply Sirona Implants

Multiple-use, stainless steel, non-sterile

| | Astra Tech Implant System EV | | Xive | | Ankylos | | | Ankylos | |
|-----------|--|--|--|--|--|--|--|--|--|
| | | | Note: For finger force only | | *Compatible with Xive | | | Note: For finger force only | |
| |  |  |  |  |  |  |  |  |  |
| Order no. | 25776 | 25777 | 26-6120 | 46-5750 | 3103 7510 | 3103 7511* | 3103 7512* | 3103 3410 | 3103 3415 |

